



INFORMATION

on the use of the Dallmer adhesive dispenser



Preparation

We recommend whilst you are preparing the hoof, you put the shoe **and** the tubes of adhesive in a carton and use your hot air-gun to warm them to about 38-40 C. **Don't heat the mixing tube !**



Opening the tubes

Open at the line marked on the neck with pincers or a sharp knife.

Check the open end

Before putting the mixing tube onto the tube, press gently to expel a couple of drops of **each** material.

Put on the mixing tube

The patented Dallmer mixer-dispenser delivers the correct combination of resin and hardener. Place the mixing tube onto the tube and twist about half a turn clockwise.

Gently push the piston and dispense a wide band of adhesive onto the fleece lining of the shoe.



Close the tube

If there is some adhesive left over, unscrew the mixing tube and clean out. A cap is provided for the end.

