



**Sole-Guard™** (REF#s 46160) is a fast setting, 30 second, liquid urethane hoof protection material providing a durable, shock absorbent, support that retains its shape and flexibility indefinitely. Sole-Guard is designed for use without shoes and adheres to the sole sealing out moisture and debris, protecting both the frog and sole. Sole-Guard will leave the sole in excellent condition.

#### PLEASE NOTE

- **Vettec products are best used between 65° and 85° F (18° to 29° C).**
- Areas of wetness or bleeding should not be covered.
- Use of disposable gloves is recommended.
- **Do not burn or “Hot Shoe” this product.**
- **Use only Vettec mixing tips.**

#### CARTRIDGE PREPARATION

1. To open the cartridge, remove the tip at line with nippers.
2. Before attaching the mixing tip, insert cartridge into gun, label side up and equalize the cartridge by squeezing out a small amount to ensure that both sides are dispensing evenly.
3. Attach mixing tip by pressing and twisting clockwise.
4. For additional applications, remove old tip, re-equalize the cartridge, and attach a cartridge cap.

#### HOOF PREPARATION

1. Trim and balance the hoof and conservatively pare the sole. With hoof knife trim and clean the frog, bars and commissuers.
2. **DRY HOOF THOROUGHLY** using a heat gun, even if you don't think it needs it. Don't overheat.

#### PROCEDURE 1 – When working on level surfaces

1. Immediately after drying, hold the hoof as flat and level as possible.
2. With the frog turned upward, dispense Sole-Guard and fill in the sole until the desired level is achieved.
3. This is usually just short of the white line. **Do not fill above the level of the wall.**
4. Place plastic sheet over sole and smooth to desired effect.
5. Hold the hoof for approximately 1 minute, then release.
6. Remove plastic sheet in approximately 5-6 minutes.

#### PROCEDURE 2 – When working on soft or uneven surfaces

1. After drying; hold the hoof and dispense Sole-Guard allowing material to flow towards toe.
2. Apply a foam board to the toe region first and lay the board towards the heel leveling the Sole-Guard material.
3. Hold for one minute and set the foot down.
4. Remove foam board after 6 minutes.

#### EMERGENCY/TRAIL RIDING PROCEDURES

1. Wire-brush the sole and frog removing all dirt and debris.
2. Dry the sole and frog using clean rag or towel.
3. Apply as above using either method 1 or 2.

**Limited Remedy:** Follow instructions carefully. If your application is not covered by these instructions, call 800/483-8832 before using this product. Buyer agrees that if Vettec products are proved defective, the user's exclusive remedy will be, at manufacturer's election, to replace the defective Vettec product or to refund the purchase price for the defective quantity. Manufacturer is not liable for direct, indirect, incidental or consequential damages regardless of legal theory asserted, including assertion of negligence and strict liability.

© 2006

Vettec Inc., 600 East Hueneme Road, Oxnard, CA 93033 USA  
800/4Vettec (483-8832) or (805) 488-3162 • Fax: (800) 444-5170 • [www.vettec.com](http://www.vettec.com)