



Super Fast™ (REF#46140) is a fast setting urethane material that is used to fabricate both durable custom shoes and foal extensions directly on the hoof.

PLEASE NOTE

- Vettec products are best used between 65° and 85° F (18° to 29° C). SEE: Cold & Hot Conditions
- Areas of wetness, bleeding, or infection should not be covered. Once healed, commence using the product.
- Use of disposable gloves is recommended.
- Do not burn or “Hot Shoe” this product.
- Use only Vettec mixing tips.

CARTRIDGE PREPARATION

1. To open the cartridge, remove the tip at line with nippers. Insert cartridge into dispensing gun, **label side up**.
2. Before attaching the mixing tip, equalize the cartridge by squeezing out a small amount to ensure that both sides are dispensing evenly.
3. Attach a mixing tip by pressing and twisting clockwise.
4. **When finished, don't release the pressure with the lever on the back of the dispenser. Remove the mixing tip immediately, discard, and replace with a cap. Push the pressure release lever.** Caps are available from your dealer.
5. For additional applications, re-equalize the cartridge, and attach a new tip.

HOOF PREPARATION

1. Trim, rasp, and level hoof as normal. Remove any weak or soft material, especially at the white line area.
2. **DRY HOOF THOROUGHLY** using a heat gun or hair dryer, even if you don't think it needs it. Do not overheat.
NOTE: ONLY if the hoof is oily, wipe the affected areas with a cloth moistened with **Denatured Alcohol**, being careful not to saturate the hoof. Dry thoroughly. **DO NOT USE ACETONE!**

CUSTOM SHOES:

1. Hold hoof upright and dispense Super Fast where your normal shoe placement would be. The material should be approximately the width of a normal keg shoe by 1/2 inch thick and extending 1 inch up the hoof wall.
2. Wait until Super Fast has started to cool down before releasing the hoof, about 2 1/2 to 3 minutes. The material will be hard before then, but won't be completely cured.
3. Rasp and trim material to the desired shoe size.
4. Additional layers can be applied for corrective shoeing or changing heel angles. If the previous layer has become contaminated, rasp off the dirt and add more Super Fast.
5. On slippery surfaces, rasp grooves on bottom for added traction.

FOAL EXTENSIONS:

NOTE: Foals less than 30 days old; Do not bring material too far up onto the hoof wall as this could constrict the growth of the hoof. Additionally, do not lock the heels with the material.

1. **Do not apply only to the dorsal wall.** First, hold hoof up and dispense a 1/4 inch layer of Super Fast around white line area (for ground contact) and then the hoof wall. Hold hoof for one to two minutes and then set down. **The bond is to two surfaces.**
2. Apply additional layers to the dorsal wall until the desired shape and length is achieved.
3. Rasp and trim material as desired.

COLD CONDITIONS (Below 65° F)

- Store the cartridges in an insulated “cooler” warmed with microwaveable heat pads or 1 to 2 tightly capped gallon jugs filled with hot water.

HOT CONDITIONS (Above 85° F)

- **Super Fast** will set very quickly. To retard the set time, store the material in an insulated cooler with 1 or 2 freezer packs.

Limited Remedy: Follow instructions carefully. If your application is not covered by these instructions, call 800/483-8832 before using this product. Buyer agrees that if Vettec products are proved defective, the user's exclusive remedy will be, at manufacturer's election, to replace the defective Vettec product or to refund the purchase price for the defective quantity. Manufacturer is not liable for direct, indirect, incidental or consequential damages regardless of legal theory asserted, including assertion of negligence and strict liability.

© 2003 rev 08/24/05

Vettec Inc., 600 East Hueneme Road, Oxnard, CA 93033 USA
800/4Vettec (483-8832) or (805) 488-3162 • Fax: (800) 444-5170 • www.vettec.com