




Instructions for Applying the Dalric B0, B1, and B2 Cuffs

State of the Art Treatment for Club Feet in Young Foals

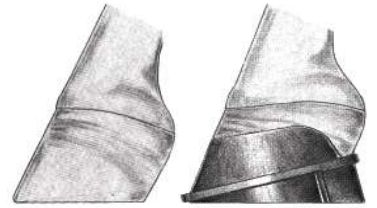

FOOT PREPARATION

- Trim and balance the foot.
- Round the hoof ground surface to accommodate the seating of the foot into the cuff.
- Roughen the wall slightly with the smooth side of the rasp to provide an adequate glue contact surface.
- Thoroughly clean the dirt from all the hoof cracks and cover potential abscesses with a small piece of tape or Keratex Putty to prevent sealing any bacteria under the glue.
- Fit the shoe to the foot.
- Using the 8/32" bolts provided, attach the heels of the shoe to the wedge.
- When the foot has a steeper angle than the shoe, simply slice the front of the shoe at the seam to accommodate a proper fit.

SHOE APPLICATION

- If you are working with an animal that is fractious, you may have your vet administer a sedative.
- Clean the hoof wall with denatured alcohol and ensure the foot is dry. Use a heat gun if necessary.
- Open the **NANRIC Mega Bond™** package and unscrew the locking collar and remove the storage cap. Attach a mixing tip by pressing on to the open cartridge. Secure the locking collar over the top of the attached tip with a quarter turn. Insert the cartridge into the dispensing gun and dispense a small amount through the mixing tip to equalize the cartridge.
- Scan the QR code on the back of the cartridge for more detailed adhesive instructions.
- Apply **NANRIC Mega Bond™** to the felt side of the shoe only.
- Apply the shoe snugly to the clean foot.
- **Mega Bond™ sets in 30 seconds, so remember to work quickly.** Total cure time is 2-3 minutes. If you need a longer set time, Mega Bond™ can be stored in a refrigerator prior to use.
- To apply a second shoe, simply apply a new tip to the Mega Bond™ cartridge.
- When finished, apply the storage plug to the Mega Bond™. The remaining adhesive can be used at another time.
- This shoe may be used without the permanent wedge if you are simply using the shoe for the protection of broken toes. In this instance, use the wedge for application only, and remove it in 30 minutes.

SHOE REMOVAL

- Use nippers or a rasp to separate the entire cuff from the horn wall and then pull the shoe.

RECOMMENDED SHOETING SCHEDULE

- For the best results, follow this shoeing schedule as the foal's feet continue to grow. Exceeding the recommended shoeing time can alter natural foot development.

Age	Maximum Time for Shoes To Be Worn	Days Following Shoe Removal To Re-evaluate
Birth - 2 Weeks	4 Days	7 Days
2-4 Weeks	7-10 Days	7-10 Days
4-8 Weeks	10-14 Days	7-10 Days
8-12 Weeks	14 Days	7-10 Days
3 Months	21 Days	7-10 Days

SORENESS AFTER APPLICATION

- Fitting the shoe too tightly will pinch the quarters causing slight soreness. If this occurs, pull the shoe and reapply after the foal is sound.
- Sealing bacteria in broken areas along the white line will cause acute lameness. If this occurs, pull the shoe immediately and notify your veterinarian. The abscess will need to be opened and appropriate therapy must be administered.
- Be careful when working with club feet. The toes are often worn very thin, which produces a fragile white line.